



Cystoscopy Simplified
QUICK · EASY · EFFECTIVE



The CystoSure[®] Advantage


Now there is a fast, convenient way to conduct cystoscopy effectively. CystoSure[®] Plus combines the familiarity of a urinary catheter with the functionality of a cystoscope into a singular product providing reliable and consistent results.

www.CystoSure.com


Introducing CystoSure® Plus

a simple all-in-one silicone urinary catheter with an additional port for cystoscopy. This unique and patented system is quick to insert, easy to learn, and effective in providing you the visualization you need for assessment of the female urinary bladder.


Simple & Standardized Steps

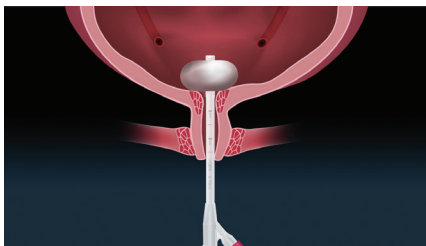


Insert CystoSure® Plus catheter and inflate balloon 5cc.

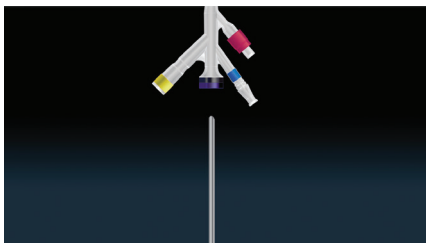

Insert CystoSure® Cystoscope into purple port.

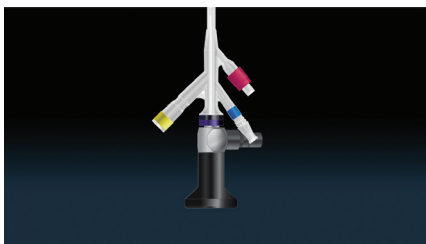

Push cystoscope forward until it stops.

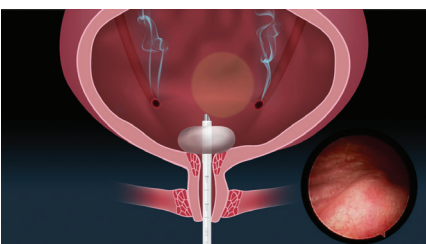

Rotate cystoscope 360° for complete view of bladder and ureteral openings.

CystoSure® Cystoscope

30° & 70° Lens

CystoSure® Plus Silicone Access Catheter

Smooth rounded tip to reduce irritation

Patented Pancake Balloon ideally positioning scope for maximum viewing of the bladder


Specifications

CystoSure® Plus Length	263 mm
Shaft Outer Diameter	16F
Lumen Inner Diameter	3.1mm
Material	100% Flexible Silicone
Pancake Balloon size	5 ml (cc)

The CystoSure® Plus Silicone Access Catheter is indicated for use with the CystoSure® Cystoscope.

Specifications

Working Length	270 mm
Outer Diameter	2.7 mm
Direction of View	30 degrees, 70 degrees

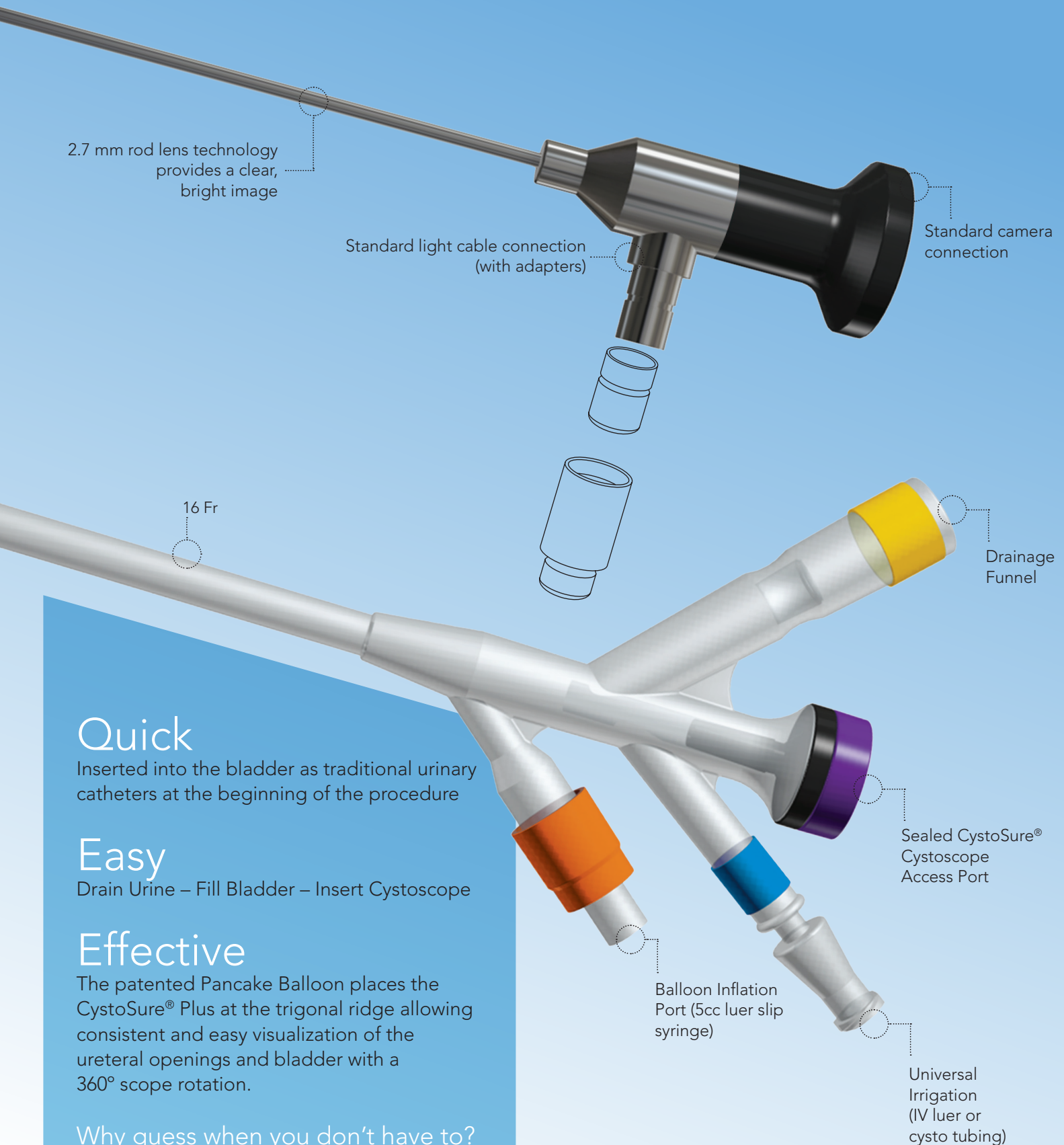

2.7 mm rod lens technology provides a clear, bright image

Standard light cable connection (with adapters)

Standard camera connection

16 Fr

Drainage Funnel

Quick

Inserted into the bladder as traditional urinary catheters at the beginning of the procedure

Easy

Drain Urine – Fill Bladder – Insert Cystoscope

Effective

The patented Pancake Balloon places the CystoSure® Plus at the trigonal ridge allowing consistent and easy visualization of the ureteral openings and bladder with a 360° scope rotation.

Why guess when you don't have to?

Sealed CystoSure® Cystoscope Access Port

Balloon Inflation Port (5cc luer slip syringe)

Universal Irrigation (IV luer or cysto tubing)

Why CystoSure® Plus?


Enabling Routine Cystoscopy

In the OR

- QUICK Single Insertion reducing catheter and instrument exchanges, saving time and reducing chance of infection from multiple exchanges
- EASY to learn and similar to other silicone drainage catheters
- EFFECTIVE providing a simplified system to conduct routine cystoscopy, without the need to purchase and maintain complex cystoscopy trays

In the Office

- QUICK Single Insertion eliminating catheter and instrument exchanges, saving time
- EASY to learn, and incorporate into your office practice
- EFFECTIVE and efficient in providing you and the patient the option to conduct Comprehensive Bladder Diagnostics in a SINGLE VISIT

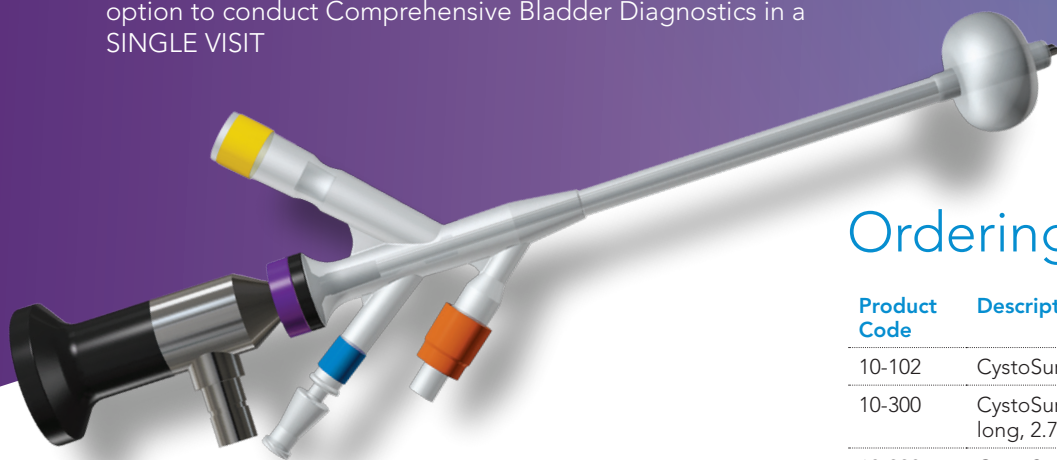

Ordering Information

Product Code	Description	Unit of Measure
10-102	CystoSure® Scope Sterilization Tray	1
10-300	CystoSure® Rod Lens Cystoscope, 270 mm long, 2.7 mm OD, 70 degrees	1
10-330	CystoSure® Rod Lens Cystoscope, 270 mm long, 2.7 mm OD, 30 degrees	1
10-400	CystoSure® Plus Silicone Access Catheter, 263 mm long, 16F	5 / Box

To place an order, contact Emmy Medical at:

phone 775-800-7300

fax 844-225-4600

email cs@cystosure.com

Peace of Mind

FOR YOU · FOR YOUR PATIENTS · FOR YOUR HOSPITAL



www.CystoSure.com

Emmy Medical, LLC. All rights reserved. Specifications are subject to change without prior notice. Cystosure® is a registered trademark of Emmy Medical, LLC. FDA Clearance K142194. ADV-0023 Rev A

